

**Contour**  
Evolving with you

# GlucoContro.online

## Connect with your patients' data remotely\*



To assist with diabetes management, invite your patients to share their blood glucose results.

[www.glucocontro.online](http://www.glucocontro.online)

\*Patients need to be registered in GlucoContro.online and approve sharing their data with their healthcare professional

  
**ASCENSIA**  
Diabetes Care

# Easy access to patient information

Available on one screen with a single click



**Fast and easy access**  
to CONTOUR®DIABETES  
app data stored in  
CONTOUR®CLOUD.



Panel with **general statistics** and **complete list of registered patients**



**Personalized information**  
about each registered **patient** (target ranges, time frames)



*Patient and physician name included for example purposes*



**Generate reports**  
(Including AGP report\*, PDF, print)




**Quick data summary**  
traffic light color coded charts



**A secure environment**  
for your patients to share information with you



**Download results from CONTOUR® meters**  
without system administrator permissions

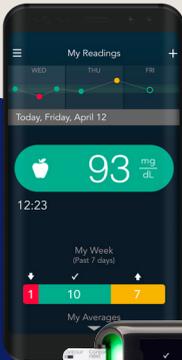

\*Ambulatory glucose profile

# A new way to view blood glucose results

Provides understanding of glucose values thanks to:

## Quick access menu



|  |                                       |                   |                  |                                  |                              |                                  |           |
|--|---------------------------------------|-------------------|------------------|----------------------------------|------------------------------|----------------------------------|-----------|
| Readings 16<br>meas. Per day 5.33<br>w/o meas. 0 | ⚠️ 7%<br>Glucose Management Indicator | ⚠️ 3<br>Very High | ⚠️ 1<br>Very Low | ⚠️ 78mg/dL<br>Standard Deviation | 156mg/dL<br>Average pre-meal | ⚠️ 245mg/dL<br>Average post-meal | 45%<br>CV |
|--|---------------------------------------|-------------------|------------------|----------------------------------|------------------------------|----------------------------------|-----------|

**Summary:** Relevant information for decision making



**Glucose concentration ranges:**  
To view the percentage of readings in each glucose concentration range



**Glucose concentration distribution:**  
View variability of glucose readings

Daily Total Insuline/Carbs/Activity

Sunday 14/03

Avg. 158.20 mg/dL

10.0 U

0.0 g

0 min

|            |              |              |              |             |
|------------|--------------|--------------|--------------|-------------|
| 67<br>8:52 | 188<br>12:11 | 280<br>14:53 | 183<br>18:33 | 83<br>22:03 |
|------------|--------------|--------------|--------------|-------------|

**Measurements table:** Simple display of all collected data for each day

|             | Total count | Average   | Very High | Very Low | High | Target | Low | Std. dev. |
|-------------|-------------|-----------|-----------|----------|------|--------|-----|-----------|
| 🕒 Fasting   | 16          | 171 mg/dL | 8%        | 3%       | 50%  | 38%    | 13% | 78 mg/dL  |
| 🕒 Pre-meal  | 3           | 68 mg/dL  | 0%        | 33%      | 0%   | 33%    | 67% | 28 mg/dL  |
| 🕒 Post-meal | 3           | 156 mg/dL | 0%        | 0%       | 67%  | 33%    | 0%  | 43 mg/dL  |
| 🕒 No flag   | 5           | 245 mg/dL | 60%       | 0%       | 80%  | 20%    | 0%  | 62 mg/dL  |
| 🕒 All       | 5           | 167 mg/dL | 0%        | 0%       | 40%  | 60%    | 0%  | 53 mg/dL  |

**Results summary:** For a more detailed summary of patient blood glucose (BG)

## Simple onboarding and patients invitation

### Register in 3 simple steps

1. Visit [glucocontro.online](https://glucocontro.online) and click register
2. Fill out information
3. Wait for the approval from the practice administrator†

### Create an account

#### Account Type



Choose this type of an account if you have been diagnosed with diabetes.



Choose this type of an account if you are a healthcare professional.




Choose this type of an account if you want to take care of someone with diabetes but you are not a diabetic.

#### E-mail address

#### First Name

#### Last Name

### Click “Add new patient”

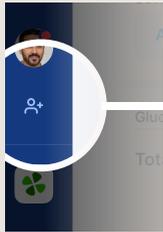

### Add new Patient

Manual QR Code

**Mandatory**

First Name


Last Name

Country

**Optional**

I want to send an e-mail invitation to this patient

**Add new Patient**

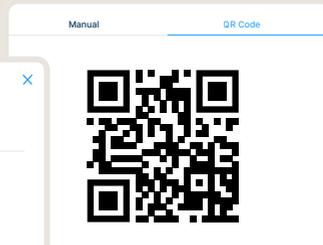

The  indicates that your patient has accepted the invitation

 **Paul Smith**   
36 y/o Type 1  
Physician: **Simon Bryant**



### Customer Service:

Phone: 1-800-348-8100

Email: [support@contournext.com](mailto:support@contournext.com)  
[ascensiadibetes.com](https://ascensiadibetes.com)

† It may take up to 24 hours

SdNcenter is the legal manufacturer and developer of GlucoContro.online and is partnering with Ascensia to market and deliver the tool to HCP and Patient users.

© 2022 Ascensia Diabetes Care. All Rights Reserved.

Ascensia, the Ascensia Diabetes Care logo, Contour, Microlet, Second-Chance, the Second-Chance logo, No Coding, and the No Coding logo are trademarks and/or registered trademarks of Ascensia Diabetes Care. Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc. Google Play and the Google Play logo are trademarks of Google Inc.

PP-GluCo-US-0007