


DMS Pro Quick Set Up

The Eversense DMS Pro software is intended for clinic use. This sheet shows a short cut for setting up a single DMSPro account for your entire clinic. This short cut is not recommended if you have multiple users in the clinic. For complete instructions, see the DMS Pro User Guide (eversensedisabetes.com).

1 Sign Up for DMS Pro

Go to <https://uspro.eversensedms.com>.
Click on **Sign Up**.

Click on **Sign-Up**

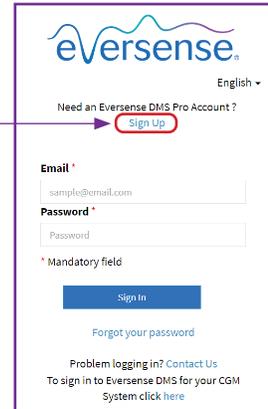

2 Request an Eversense DMS Pro Account


Complete the form by entering in your clinic's information.

- Under "First Name" and "Last Name" it is recommended to either put your clinic name, or a primary health care provider in the clinic. (This is what your patients will see when they share data.)
- Select a "Title" from the dropdown, and complete the remaining required fields.

Notes:

- If you want to add individual providers to your Eversense DMS Pro clinic, see the Eversense DMS Pro User Guide.
- You cannot use the same email address for both an Eversense DMS Pro account and a personal Eversense DMS account.

Click **Submit** when form is completed.

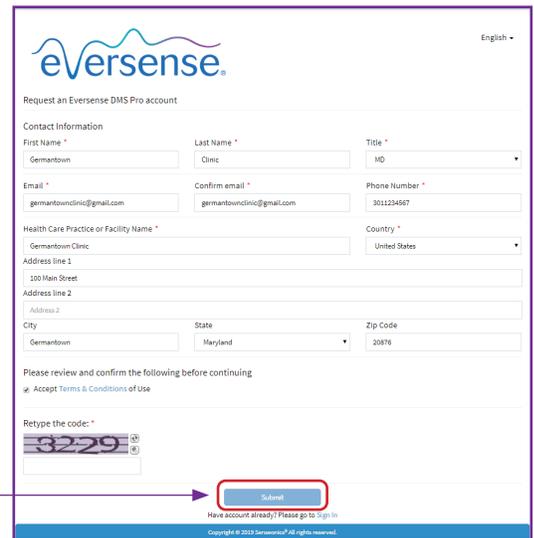

3 You Will Receive Two Emails

- Account registration confirmation.
- Account approval email (may take up to 24 hours). Click on the link* in the email to complete account activation.
- Record important account information in the box to the right.

***The link expires after 72 hours, but you can request that a new email be sent.**

Clinic Name:

Email:

Password:

Clinic ID#: (see Step 4)

4 Sign in to the clinic DMS Pro account

The assigned clinic ID is at the top.

Note: Patients will need the Clinic Number when they send a request to share their CGM data with the clinic.

eversense
English
Need an Eversense DMS Pro Account? Sign Up
Email *
sample@email.com
Password *
Password
* Mandatory field
Sign In
Forgot your password
Problem logging in? Contact Us
To sign in to Eversense DMS for your CGM System click here

XYZ Clinic - 9
List of Patients
Search Patient

5 Inviting Patients to the Clinic

- On the navigation bar on the left, click **Patient** then **Invite Patient**.
- Enter the patient's Eversense email (the one used when setting up their mobile app and creating their Eversense account).
- Click **Search Eversense**.

eversense
English
Home
Staff
Patients
Invite Patient
Pending Requests
Help

Email *
sample@email.com
Search Eversense
Send Reset

If the patient's email is registered, the field to the right will be displayed. Complete all mandatory information, then hit **Send**. This generates an invitation email to the patient.

Note: If the entered email is not registered with Eversense DMS, you will receive an error message. Confirm with the patient what email address was used when setting up their account.

Email *
sensdemo2@gmail.com
Search Eversense
User already exist in Eversense DMS
MRN
Medical Record Number
Provider *
Select...
Last Name *
Brown
First Name *
Jane
Date Of Birth
Dec 31 1989
Male Female
Send Reset

Once the patient accepts the invitation, the **X** under Data Share will turn to **✓**. Click on the patient's last name to view their reports.

Montgomery Endocrinology - 2

List of Patients
The record has been updated successfully

Search Patient Patients by Provider Export Data Show deactivated patients

Patient last name	Patient first name	MRN	Gender	DOB	Provider Name	Status	Data Share		
<u>Kasodimos</u>	SENS DEMO-Tommy	12345	Male	Jun 24, 1993	Barbara Montgomery, MD	Active	✓		

Note: Patients may request to share their data. Pending requests from patients appear in the left navigation bar. Click on **Patients > Pending Requests**.

For additional information or support on Eversense DMS Pro, refer to your DMS Pro User Guide, contact your Eversense Representative, or contact Eversense Customer Care at:

1-844-SENS4U (736-7348)
Support@eversenseddiabetes.com



Continuous Glucose Monitoring System

MANUFACTURED BY:
SENSEONICS, INC.
20451 Seneca Meadows Parkway | Germantown
MD 20876-7005 | USA
301.515.7260 | global.eversenseddiabetes.com



DISTRIBUTED BY:
ASCENSIA DIABETES CARE US, INC.
5 Wood Hollow Road | Parsippany
NJ 07054 | USA
+1.862.225.2902 | ascensia.com