




Appointment Date/Time: _____

Getting Ready for Your CGM Appointment

Congratulations on your decision to use the Eversense® Continuous Glucose Monitoring (CGM) System. Let's get you started on the Eversense journey!

Before Your Appointment

Follow the *Getting Started Steps* in the Eversense Quick Reference Guide (<https://eversensediabates.com/user-guides>)

- From your compatible mobile device, download the Eversense App from the App Store® or Google Play™
- Tap the Eversense icon  to launch the app, and follow the prompts to create your Eversense account
 - On the Welcome Screen, choose "I do not have a Smart Transmitter"
 - On the Unit of Measurement screen, leave it set at mg/dL
 - Write down your account username (email) and password:
Email _____
Password _____
- After you create your account, you will receive an email. Within 72 hours, click on the link in the email to activate your account

The Eversense® Continuous Glucose Monitoring (CGM) System is indicated for continually measuring glucose levels in persons age 18 and older with diabetes for up to 90 days. The system is indicated for use to replace fingerstick blood glucose (BG) measurements for diabetes treatment decisions. Fingerstick BG measurements are still required for calibration twice per day, and when symptoms do not match CGM information or when taking medications of the tetracycline class. The sensor insertion and removal procedures are performed by a health care provider. The Eversense CGM System is a prescription device; patients should talk to their health care provider to learn more. For important safety information, see <https://eversensediabates.com/safety-info/>.

Day of Your Appointment

- Wear a loose short-sleeved or sleeveless shirt
- Bring to the appointment:
 - Eversense account username (email) and password
 - Your mobile device, fully charged and with Eversense App installed
- Return to normal daily activities. However, to allow proper healing, it's not recommended that you soak in a tub, swim, or do strenuous activities for five days.

Questions? We're here for you! Eversense® Customer Care: 1-844-736-7348 (1-844-SENSE4U)